

CORONA TEST INSTRUCTIONS FOR USE

CORONA TEST • INSTRUCCIONES DE USO

CORONA TEST • MANUEL D'UTILISATION

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CORONA TEST • INSTRUKCE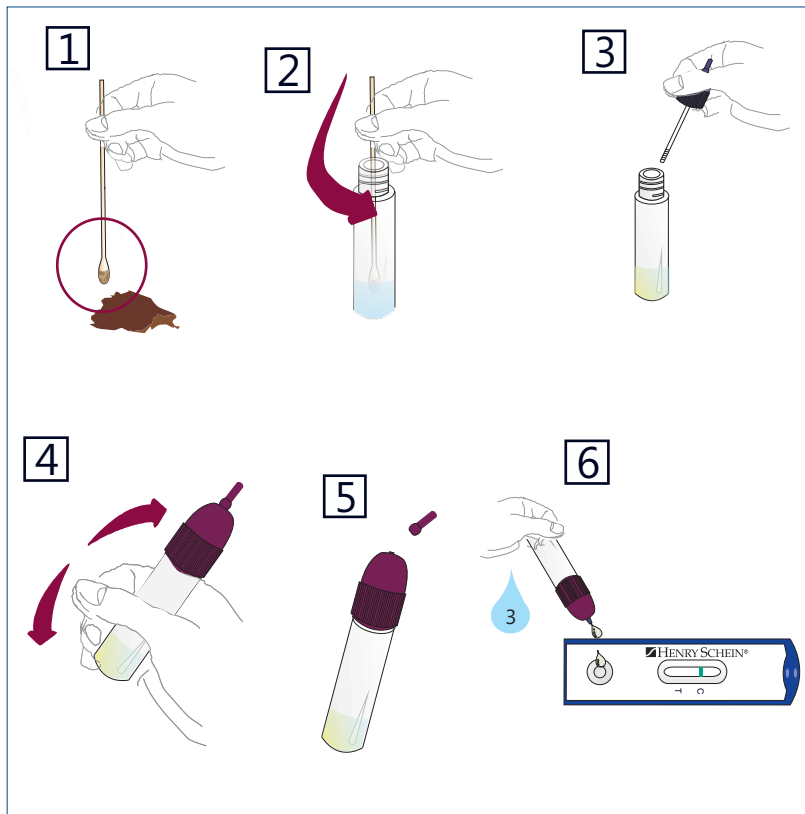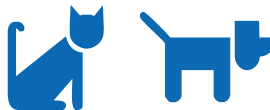

CORONA TEST • INSTRUCTION FOR USE

The Henry Schein corona test is used to test rapidly and reliably for coronavirus (CCV and FCoV) antigen in stool samples.

HANDLING THE SAMPLE

Fresh stool requires no particular pre-treatment and can be examined directly.

The samples can be cooled between 2-8°C for up to two days and stored at -20°C for several weeks.

Before the test is carried out, the samples have to be brought to room temperature. Frozen samples should be thoroughly mixed after thawing.

TESTING

1. Take a stool sample using the cotton bud. Make sure that the upper end of the cotton bud is covered.
2. Unscrew the test tube and place the cotton bud with the stool inside. Swirl the bud around in the fluid several times, so that some of the sample material can come loose.
3. Screw the test tube shut.
4. Agitate vigorously, so that the contents become well mixed and the fluid discolours.
5. Break the spike of the test tube by applying pressure sharply. Open the test cassette packaging and place the cassette on a horizontal surface.
6. By applying gentle pressure, you can apply initially three drops of the test material to the test field of the test cassette. If, after the first three drops, the fluid has not yet started to run across the test strips, add a further drop.

An excessive amount of sample feces can cause a brown line in the T-region (T), which must not be seen as a positive result. The test is invalid and should be repeated.


ANALYSIS

After 10 minutes, the test can be analysed. One or two red lines appear in the reaction field.

POSITIVE TEST RESULT:

The test line and the control line become visible.

NB: even a faint test line must be interpreted as a positive test result.

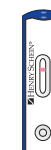NEGATIVE RESULT:

Only the control line becomes visible.

INVALID:

If no control line is shown, the test is invalid.

NB: the maximum time to reading is 20 minutes.

All results read after 20 minutes are invalid.

DISPOSAL

All materials used must be disposed of carefully, because the material could be infectious. Test materials and used cassettes must be put in plastic bags and then disposed of.

PLEASE NOTE

Only for animal use • Only for professional use • Use the cassette as soon as possible after opening • Please observe the required sample quantity • Do not apply any sample material to the reaction field • Do not touch the reaction field • Use only the fluid supplied • To avoid cross-contamination, a separate collection vessel should be used for each sample • Stool samples may be infectious, so suitable precautions should be taken for handling and disposal – avoid skin contact and wear laboratory gloves and clothing • Do not use after the expiry date • Do not use the test cassette if the packaging is damaged • Observe analysis times indicated • Store and transport the test only at the temperatures indicated

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HENRY SCHEIN U.K. HOLDINGS LTD.