FeLV & FIV TEST INSTRUCTIONS FOR USE

FeLV & FIV TEST • INSTRUCCIONES DE USO


FeLV & FIV TEST • MANUEL D'UTILISATION

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
FeLV FIV TEST • INSTRUKCE

I.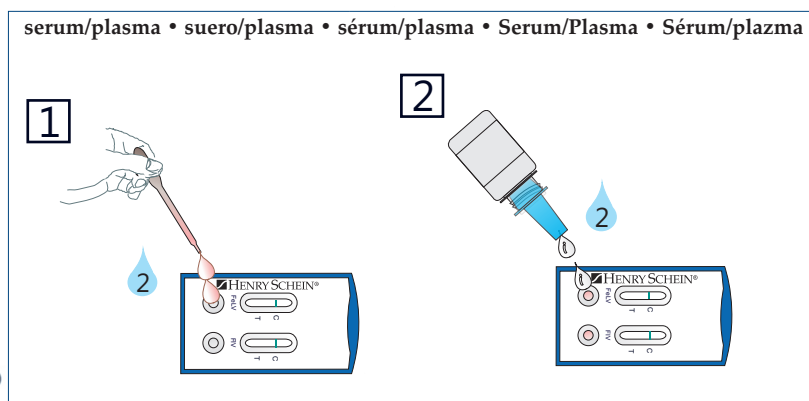

II.

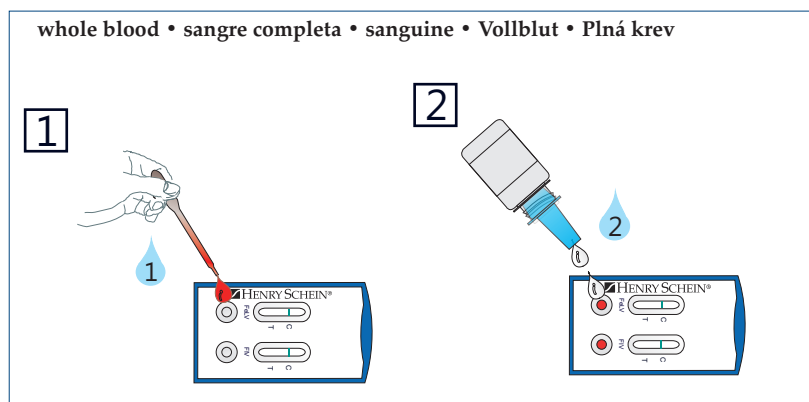SENSITIVITY AND SPECIFICITY
INSECTA ET SPERMATOPHYTA
study 2014
Bacteria 2014

FeLV & FIV Test	Enzym-linked Immunosorbent Assay	
	Sensitivity • Insecta	Specificity • Spermatophyta
FeLV	95,83 %	99,99 %
FIV	98,51 %	99,99 %

FeLV & FIV TEST • INSTRUCTION FOR USE

The Henry Schein dual FeLV and FIV rapid test is used to test for feline leucosis virus (FeLV) antigen and feline immunodeficiency virus (FIV) antibodies in full blood, serum, and plasma.

HANDLING THE SAMPLE

Do not keep the samples at room temperature for too long. Serum and plasma can be stored between 2-8°C for up to 3 days. For longer storage, the samples should be frozen at below -20°C. Venous full blood should be stored between 2-8°C, if the test is due to be carried out within 2 days from sampling. Do not freeze full blood samples. If samples are being sent somewhere, they should be packed in accordance with local regulations concerning aetiological agents. Bring the samples to room temperature before testing. Frozen samples must be fully thawed before testing and mixed well. Samples should not be refrozen and re-thawed.

General instructions for the use of blood samples:

Separate the serum or plasma as rapidly as possible from the blood, to avoid haemolysis • Heparin blood and EDTA blood can be used to extract plasma • Only use clear rather than haemolysed samples • Testing should take place immediately after sampling

I. TESTING IN THE CASE OF SERUM AND PLASMA


1. Take blood from the cat and extract the serum or plasma. Using the pipette, take up the sample material obtained. Add two drops of the sample material to the two test fields. Let the sample material soak in for a few seconds.
2. Then open the reagent vial and add two drops to each of the test fields. The fluid begins to run up the test strip. If the fluid does not begin to run up after a few seconds, add a further drop to the reagent vial.

II. TESTING FULL BLOOD

1. Take blood from the cat. Using the pipette from the test, take up the sample material obtained. Add a drop of the sample material to the two test fields. Let the sample material soak in for a few seconds.
2. Then open the reagent vial and add two drops to each of the test fields. The fluid begins to run up the test strip. If the fluid does not begin to run up after a few seconds, add a further drop to the reagent vial.

ANALYSIS

After 10 minutes, the test can be analysed. One or two red lines appear in the reaction field.

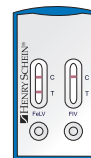POSITIVE TEST RESULT:

The test line and the control line become visible. The virus in question is demonstrated. The figure shows a positive FeLV test result.

NB: even a faint test line must be interpreted as a positive test result.

NEGATIVE RESULT:

Only the control line becomes visible.

INVALID:

If no control line is shown, the test is invalid. **NB: the maximum time to reading is 20 minutes. All results read after 20 minutes are invalid.**

DISPOSAL

All materials used must be disposed of carefully, because the material could be infectious. Test materials and used cassettes must be put in plastic bags and then disposed of.

PLEASE NOTE

Only for animal use • Only for professional use • Use the cassette as soon as possible after opening • Please observe the required sample quantity • Do not apply any sample material to the reaction field • Do not touch the reaction field • Use only the fluid supplied • Do not use after the expiry date • Do not use the test cassette if the packaging is damaged • Observe analysis times indicated • Store and transport the test only at the temperatures indicated

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HENRY SCHEIN U.K. HOLDINGS LTD.