FeLV Ag single test

10 x

#(2)800004

Rev.: KI-40-00110-01

FeLV Ag single test

Rapid test

- PT Teste para FeLV Ag
- CS FeLV Ag Single Test
- PL Test FeLV Ag
- RO Testul unic FeLV Ag
- SK FeLV Ag Single Test
- DA FeLV Ag Single Test
- NO FeLV Ag enkelt test
- SV Singeltest FeLV Ag
- ZH FeLV Ag 单项测试

FeLV Ag single test

Rapid test

- ES Test de único Ag de VLFe
- FR FeLV Ag Single Test
- DE FeLV Ag Einzeltest
- IT Test singoloFeLV Ag
- NL FeLV Ag enkele test

For use in cats

Reorder # (2)800004

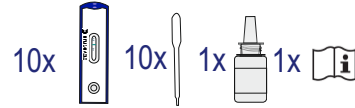
LOT 40001TTMMJJ

JJJJ-MM

30°C
2°C

Distributed by:
Covetrus BV
Beversestraat 23
5431 SL Cuijk (NL)
cbproducts@covetrus.com

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Rev.: PO-40-001-01

covetrus

FeLV Ag single test single teststrip

LOT: 40001TTMMJJ

Exp.:2022-03

Distributed by:
Covetrus BV
Beversestraat 23
5431 SL Cuijk (NL)
cbproducts@covetrus.com

FelV Ag single test

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- ES** Test de único Ag de VLFe - Instrucciones de uso
- FR** FelV Ag Single Test - Mode d'emploi
- DE** FelV Ag Einzeltest - Gebrauchsanweisung
- IT** Test singolo FelV Ag - Istruzioni per l'uso
- NL** FelV Ag enkele test - Gebruiksaanwijzing
- PT** Teste para FelV Ag - Instruções de uso
- CS** FelV Ag Single Test - Návod na použití

EN FelV Ag single test Instructions for use

The Covetrus FelV Ag rapid test is used to test for feline leucosis virus (FeLV) antigens in serum, plasma or whole blood.

Handling the sample

Do not keep the samples at room temperature for extended periods. Serum and plasma can be stored between 2-8°C for up to three days. For longer storage, the samples should be frozen at -20°C. Venous whole blood should be stored between 2-8°C. If the test is due to be carried out within two days from sampling, Do not freeze whole blood samples. Bring the samples to room temperature before testing. Frozen samples must be fully thawed before testing and mixed well. Samples should not be refrozen and rethawed.

General instructions for the use of blood samples:

Separate the serum or plasma as rapidly as possible from the blood, to avoid haemolysis. • Heparin blood and EDTA blood can be used to extract plasma. • Only use clear rather than haemolysed samples. • Testing should take place immediately after sampling.

Test procedure

Figure 1: Testing with serum and plasma

a. Draw blood sample from cat and extract the serum or plasma. Take the test pipette and draw sample material up into pipette. Add one drop of the sample material to the sample well. Wait a few seconds, then use the pipette to fully absorb the sample material.

Figure 2: Testing with whole blood

a. Draw blood sample from cat. Take the test pipette and draw sample material up into pipette. Add one drop of the sample material to the sample well. Wait up to one minute, to allow the test to fully absorb the sample material.

Analysis (Figures 3 and 4):

After 10 minutes, one or two red lines will appear in the reaction field, and the test can be analysed.

Positive test result (Figure 3):

Both test line and the control line appear in the reaction field. NB: Even a faint test line must be interpreted as a positive test result.

Negative test result (Figure 4):

Only the control line appears in the reaction field.

Invalid:

If no control line is shown, the test is invalid. NB: All test results that are read after 10 minutes are invalid.

Disposal

All materials used must be disposed of carefully, because the material could be infectious. Test materials and use cassettes must be put in plastic bags and then disposed of.

Please note

For veterinary use only. • Only for professional use. • Use the cassette as soon as possible after opening. • Please follow the procedure for required sample quantity. • Do not apply any sample material to the reaction field. • Do not touch the reaction field. • Use only the fluid supplied. • Do not use after the expiry date. • Do not use the test cassette if the packaging is damaged. • Observe analysis times indicated. Store and transport the test only at the temperatures indicated.

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